



Lock Tactical Tourniquet



Intended Use

LOCK® Tactical Tourniquet is a mechanical hemorrhage control device designed to temporarily occlude arterial blood flow in cases of traumatic extremity bleeding.

The device is intended for rapid, effective, and safe control of severe, life-threatening hemorrhage occurring in the upper or lower extremities.

WARNINGS



Apply the tourniquet approximately 5–7 cm (2–3 inches) above the wound site. Record the application time. Prolonged application may result in tissue damage

Do not use the device if the packaging is damaged or if the product shows signs of physical damage or deterioration.

If there is insufficient time to identify the exact source of bleeding, or circumstances do not permit assessment, the tourniquet may be applied as high as possible on the affected limb.

Limitation of Liability

Neither Biosys nor any person or entity acting on its behalf shall be held liable for any direct or indirect injury, damage, or loss arising from the use of this device.

Proper use of the device is the responsibility of the user. The manufacturer accepts no liability for consequences resulting from misuse, improper use, or use contrary to these Instructions for Use.

Technical Specifications

Device Type: Mechanical Tourniquet

Intended Use: Single Use

Construction: High-strength textile, polymer, and metal components

Maximum Weight: 85g

Strap Length: 90cm



Biosys Biyomedikal
Mühendislik San. ve Tic. A.Ş.

Address: Organize Sanayi Bölgesi
Mevki 23 Küme Evleri No:4
İç kapı No:1 72200 Beşiri/BATMAN

Phone Number: +90 312 444 72 18

E-mail: info@biosys.com.tr

Website: www.biosysmed.com

WARNINGS



Use the LOCK Tactical Tourniquet only for severe, uncontrolled extremity hemorrhage.

Do not use the device on the neck, torso, or anatomical regions unsuitable for tourniquet application.

The device is single-use only and must not be reused.

Do not apply the tourniquet over joints (elbow or knee) or other anatomically unsuitable locations.

The time of application must be recorded and the casualty should be transferred for advanced medical care as soon as possible.

WARNINGS



The tourniquet should not be loosened or removed except by qualified healthcare personnel. The casualty must receive medical attention as soon as possible

Store the device in a clean and dry environment.

Do not expose the device to direct sunlight.

Do not apply the tourniquet over hard objects located inside or on clothing. Keep the device readily accessible for emergency use.

Dispose of the device according to applicable biohazard waste procedures after use.



CE Mark



Date of Manufacturer



Manufacturer



Use-By Date



Medical Device



Single Use Only



Catalogue Number



Unique Device Identifier



Batch Code



Caution



Consult Instruction for Use


1

Wrap the strap tightly around the limb.

A properly tightened strap should allow no more than three fingers to fit between the strap and the skin.


Once the strap is positioned correctly, fasten it securely.

The strap is considered sufficiently tight when your fingers cannot be inserted between the strap and the limb.

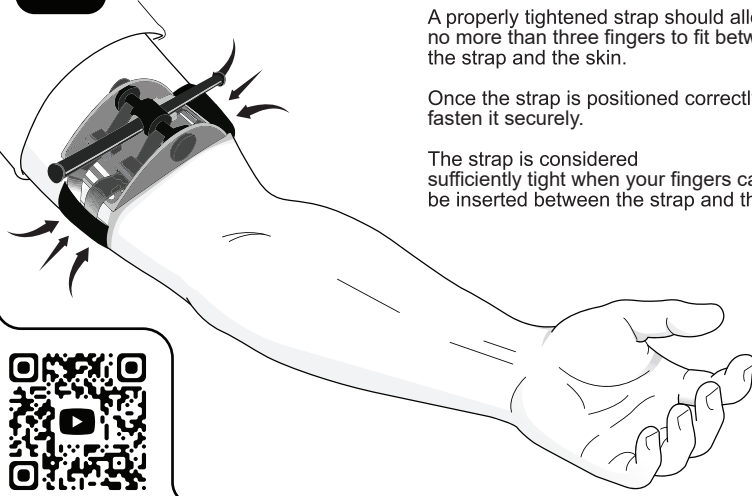


2

Rotate the windlass handle.

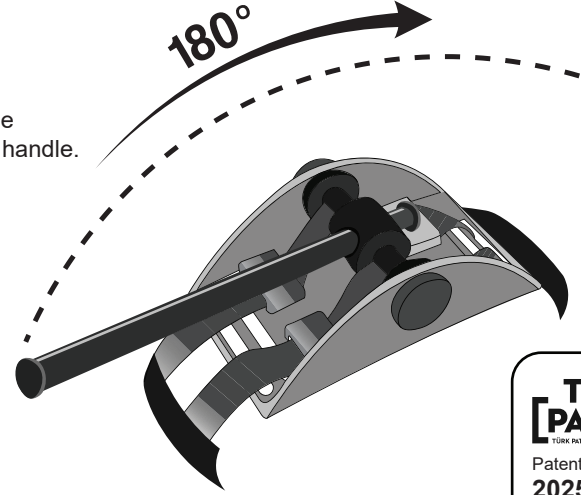

**TÜRK
[PATENT]**
TÜRK PATENT VE HAKKIA KURUMU
Patent No:
2025/013018

3

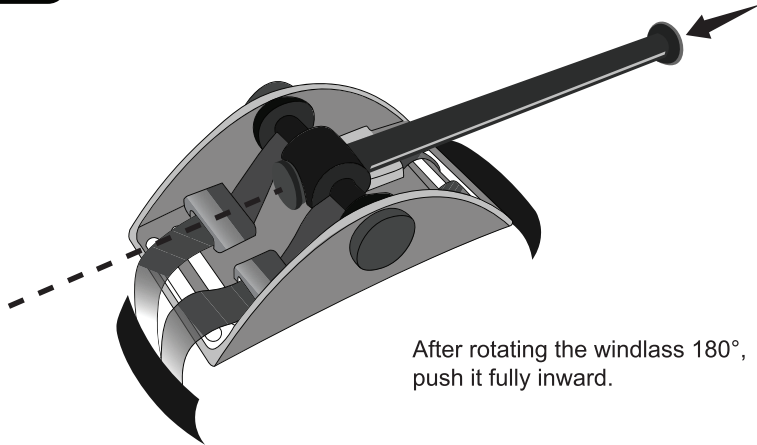


After rotating the windlass 180°, push it fully inward.

4

Continue tightening until the bleeding stops and the distal pulse is no longer detectable.

No additional locking action is required.

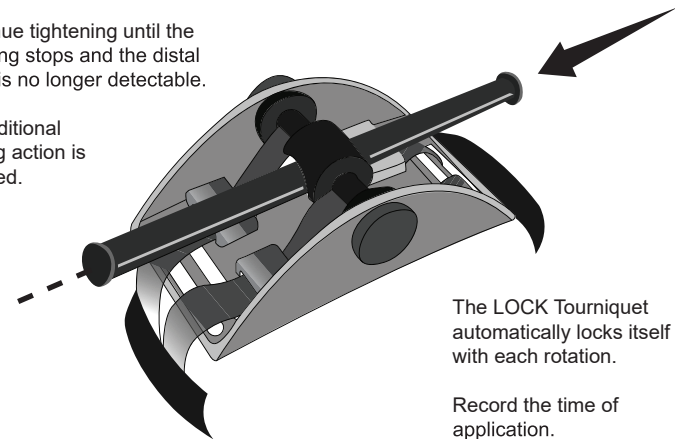

The LOCK Tourniquet automatically locks itself with each rotation.

Record the time of application.